Kelo-cote®

Advanced formula scar gel

Kelo-cote® is a lightweight, self-drying silicone gel for the treatment of scars. Kelo-cote® rapidly dries to form a sheet; this Silicone Gel Sheet layer is gas permeable, flexible and waterproof.

Kelo-cote® forms a bond with the stratum corneum (the outer layer of dead skin cells) forming a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through reduced collagen synthesis cycles, and improves the physiological and cosmetic appearance of the scar.

Kelo-cote® has been shown to flatten, soften and smooth scars, relieve the itching and discomfort of scars, as well as reduce the discoloration associated with scars.

Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars, and keloids. The 2002 International Clinical Recommendations on Scar Management recommended "silicone gels should be the first line of treatment in the initial management of scars and in the prevention of hypertrophic scars and keloids."*

Kelo-cote® can be used on joints, flexures, hairy areas and large area scars such as those caused from burns or major trauma. Kelo-cote® can be used in conjunction with pressure garments or other scar therapy options.

Intended use

Kelo-cote® Advanced Formula Scar Gel is intended for the management of old and new scars, including hypertrophic and keloid scars, resulting from surgeries, trauma, wounds and burns.

Kelo-cote® is suitable for use in children and people with sensitive skin.

Directions for opening

On first use remove the cap, cut the tip of the tube using scissors and close with the cap after use.

Directions for use

- Ensure that the affected area is clean and dry.
- Apply Kelo-cote® to the area as a very thin layer and allow to dry.
- Apply Kelo-cote® twice daily on a clean, dry skin.
- For maximum effect, Kelo-cote® should have 24 hour contact with the skin.
- Once dry, Kelo-cote® can be covered with pressure garments, sunscreen or cosmetics.

If Kelo-cote® has not dried within 4–5 minutes, you have probably used too much. Gently remove the excess and allow the drying process to continue. Recommended minimum treatment time is 60–90 days. It may take longer than 90 days for older and larger scars to achieve full benefit. Extended use is recommended if continued improvement is seen.

Tips for drying

Kelo-cote's® drying process is assisted by the temperature difference between the gel and the skin. To reduce drying time in hotter climates, keep Kelo-cote® in the refrigerator.

Caution

Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds.

Kelo-cote® should not be used on dermatological conditions that disrupt the integrity of the skin.

Kelo-cote® may stain clothing if it is not completely dry.

For external use only.

If irritation occurs, discontinue use and consult your physician.

Keep this product out of the reach of children.

Store below 25°C.

Do not use after the expiration (EXP) date printed on the tube.

Ingredients

Polysiloxanes, Silicon Dioxide

Medical device registration number: 2140001

Manufacturer:

Advanced Bio-Technologies, Inc.

Suwanee, GA 30024, USA

www.kelocote.com

Device Authorized Representative Barclay Phelps, London,

N22 8HH C€

US Patent 5,741,509

International Patents

Registration holder:

Rafa Laboratories Ltd., P.O.Box 405, Jerusalem 91003

Tel. 02-5893939

*Mustoe TA et al. Plast Reconstructive Surg 2002; 110:560-71

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