Multi-Gyn® LIQUIGEL

Read this entire information leaflet carefully before each time you use Multi-Gyn LiquiGel and keep for reference

Manufactured by:

 BioClin BV, Delftechpark 55,
2628 XJ Delft, The Netherlands www.multi-gyn.com

Read this entire information leaflet carefully before each time you use Multi-Gyn LiquiGel and keep for reference

Table of Contents

- A. Multi-Gyn LiquiGel – Treats and prevents vaginal dryness
- B. How Multi-Gyn LiquiGel works
- C. Indications
- D. Directions for use
- E. Ingredients of Multi-Gyn LiquiGel
- F. Side effects
- G. Interference with medications
- H. Pregnancy and lactation
- I. Storage
- J. Background information

A. Multi-Gyn LiquiGel – Treats and prevents vaginal dryness

- Instantly relieves vaginal dryness and stimulates the natural moisture
- Optimises the condition of vaginal tissue
- Prevents irritation and is an excellent lubricant
- Is natural, safe and protects the vaginal flora

B. How Multi-Gyn LiquiGel works

Multi-Gyn LiquiGel offers direct relief of vaginal dryness. It optimizes the condition of the vaginal tissue and stimulates its own natural moisture. Multi-Gyn LiquiGel is based on the patented 2QR-complex. This natural substance is derived from medicinal plants and consists of bio-active polysaccharides. It has the unique ability to neutralize harmful microorganisms in a safe and effective way. It is therefore also suitable to support the natural healing process of tissue damages in the intimate area (e.g. after childbirth or after an operation). Multi-Gyn LiquiGel optimizes the condition of the vaginal tissue, reduces swelling and helps to prevent infection.

It does not affect the natural vaginal flora. Multi-Gyn LiquiGel directly soothes, itch and irritations. Multi-Gyn LiquiGel is an excellent lubricant, enhancing sexual pleasure and facilitating intimate contact (such as insertion of tampons). It is also suitable for professional use in clinical examinations by e.g. gynecologist and midwives.

Regular application of LiquiGel reduces irritation and supports an optimal condition of the vagina.

The formulation is natural, safe and gentle because it is not based on toxic substances or harsh chemicals and therefore it does not have negative side-effects

Multi-Gyn LiquiGel is registered as a Medical Device Class I.

C. Indications

- vaginal dryness during and after menopause
- vaginal dryness caused by hormonal changes, medication and stress
- vaginal dryness during sexual intercourse and insertion of tampons
- vaginal atrophy, irritation, discomfort

D. Directions for use

Wash your hands and apply Multi-Gyn LiquiGel with your fingertips on the labia, in and around the opening of the vagina. It can be applied anytime you like and as often as you need its comfort.

The gel does not leave stains on your underwear. It is safe and gentle and will give you direct relief of vaginal dryness, irritations and itch.

Please note! Multi-Gyn LiquiGel is an active product that is aimed at moisturizing the intimate area, conditioning and stimulating natural moisture of the tissues. As an active product Multi-Gyn LiquiGel can give you a warm, tingling feeling after application. This is completely normal and will disappear after a few minutes.

E. Ingredients of Multi-Gyn LiquiGel

LiquiGel contains over 25% of the 2QR-complex. Other components are plant glycerin, the natural thickener Xanthan Gum and the natural "cell fluids" regulator (osmoregulator) Betaine.

Multi-Gyn LiquiGel does not contain preservatives, hormones, silicones, ingredients of animal origin, harsh chemical substances or fragrances and is completely harmless. It is not greasy, does not leave stains and is water-soluble. It does not affect tampons or condoms.

Active ingredients: Galactoarabinan Polyglucuronic Acid Crosspolymer*, Glycerin, Betaine, Xanthan Gum *2QR, patented bio-active polysaccharides derived from Aloe Barbadensis inner leaf gel by molecular filtration process.

Allergies to the ingredients of Multi-Gyn LiquiGel are very rare. We advise, when you have a known allergy to herbal substances, to try a little bit of Multi-Gyn LiquiGel on the inside of the forearm. When an itching red spot appears, allergy can be concluded to one of the ingredients in Multi-Gyn LiquiGel. In this case we advise you not to use Multi-Gyn LiquiGel.

F. Side effects

Multi-Gyn LiquiGel has no known side effects and is a safe and natural formulation. Multi-Gyn LiquiGel is not a contraceptive and contains no spermicide. However, influence on the activity of sperm cells has not been researched.

G. Interference with medications

Multi-Gyn LiquiGel does not interfere with medications.

H. Pregnancy and lactation

There are no known contraindications for the use of Multi-Gyn LiquiGel during pregnancy and breast-feeding.

I. Storage

Temperatures lower than 10°C and higher than 25°C are disadvised. Room temperature is advised.


J. Background information:

Vaginal dryness and discomforts occur frequently. Vaginal dryness and discomfort affect over half of women in all ages and stages of life. It is especially a naturally occurring problem of women in and after their menopause because of the hormonal changes. It is also the second most common discomfort of all sexually active women. Many women first notice their vaginal dryness when intercourse becomes uncomfortable, even painful. It can undermine self-confidence, moods and interfere with normal daily activities and cause a general sense of discomfort.

The causes of vaginal dryness

The most common cause is a decrease in natural vaginal fluids due to hormonal changes, such as in the second half of the menstrual cycle or linked to menopause. But there are many other factors as well that can cause or increase the problem, such as: use of hormonal contraceptives or certain medications, soap, tampons, stress, fatigue, pregnancy and childbirth, and also intercourse without proper lubrication by your natural fluids or a suitable product such as LiquiGel.

Always make an appointment with your physician or gynecologist when your complaints are persisting.

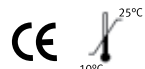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